D

Liebe Kundinnen & Kunden,

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GB

Dear Customers,

By choosing CamoCup® you have decided on the ideal mug for all who must learn to drink. CamoCup® is suitable for infants, little children, young people, people with disabilities, people with swallowing difficulties or irritations, stroke patients, old people in geriatric care, and ALL other people who enjoy the CamoCup®. **Please read the following remarks and recommendations carefully prior to use of the CamoCup®, and keep them safe for further reference.**

INFORMATION REGARDING THIS PRODUCT:

- Clean the CamoCup® prior to use as well as afterwards
- Suitable for dishwashers
- Can be sterilised / disinfected (microwave & vapour)
- Food safe
- Plastics, free of PVC, phthalates, and bisphenol A
- Developed and produced in Germany.

CamoCup® is not a toy, and it may only be used when supervised!

INSTRUCTIONS FOR USE:

- Start with the smallest amount possible (maximum one sip) in order to avoid swallowing wrong.
- In case of heavy dysphagia, only try drinking with a therapist, doctor, or other experts.
- Do not pour very cold or very hot beverages / food into the cup since temperature perception is a very subjective judgment.
- Do not heat beverages in the CamoCup® with a microwave.

- Always control the temperature prior to feeding or drinking.
- Do not use when lying flat, only when half-reclined or sitting.
- Makes it easy to administer medicine (e. g. crushed pills or syrups).
- Purposeful drinking is possible due to the unusual form of the cup and its groove.
- The open form provides the possibility to see and smell the liquid / porridge prior to drinking; this stimulates the swallowing reflex.
- CamoCup® is ideal to offer (breast) milk to the infants if breastfeeding is not possible.

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